



Patent acquisition finished  
in Japan, United States, China.

Medical device for emergency hemostasis

# J-PAD

Medical device notification number 38B1X00001100004

## No Drugs or Chemicals

No drugs or medications or chemicals are used to stop bleeding, therefore there is less risk of an allergic reaction.

## Protection of the wound

Rebleeding is not caused when the pad was removed from the wound. Because the pad maintains the wet environment of the wound.

## Quick hemostasis

Quick hemostasis is possible.  
(comparison with gauze)

## Protection from secondary infection

By the adoption of the waterproof backseat, the body fluid of the injured person does not touch the rescuer directly.

## Simple handling

Cover a wound with the pad and fix it with bandage or tape. The technical knowledge is unnecessary.



## Inexpensive

A price is lower than other emergency hemostasis articles.


## Hygienic & Reliable

The pad is sterilized, therefore there are no hygienic issues, and is approved as a medical equipment in Japan.

Made in Japan

# Structure of J-PAD

■ Three unique laminar structure to stop bleeding by causing a blood clot on a wound.

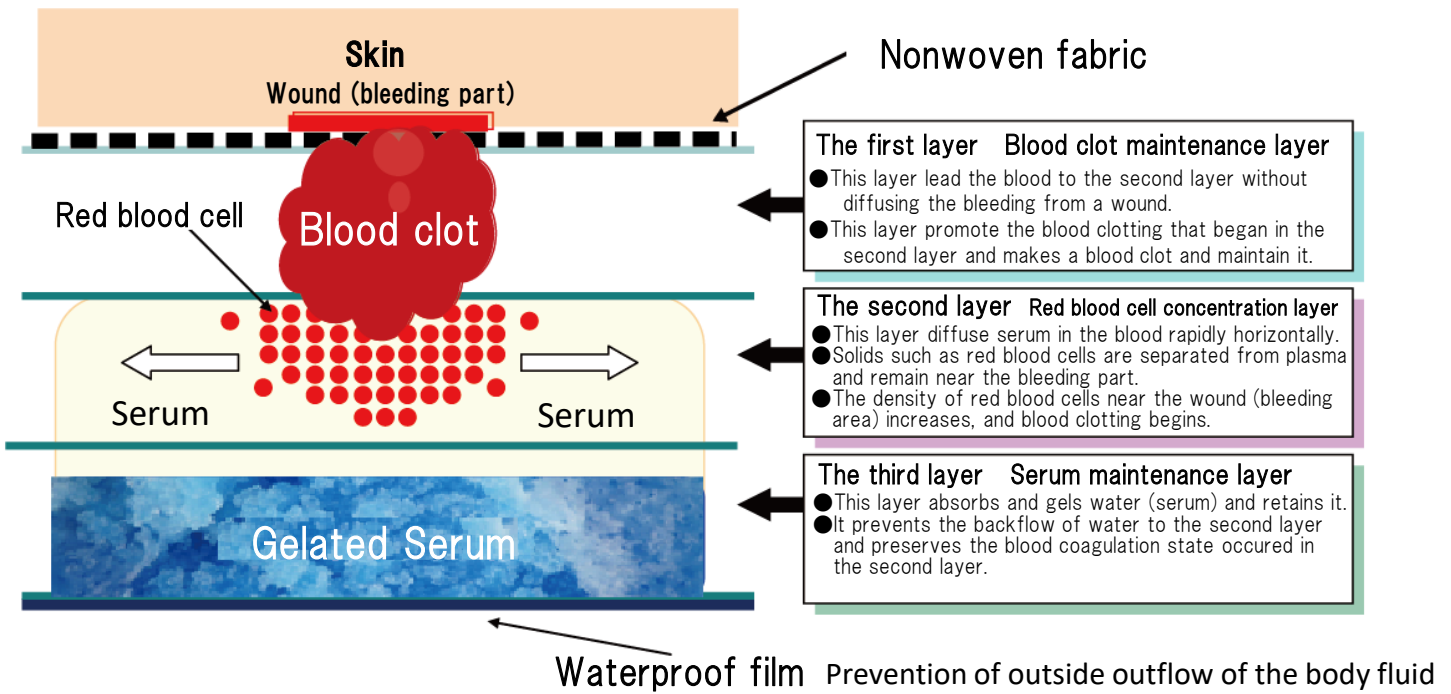

## Product specification

### Aluminum outer bag



### Sterilization bag



- Dimensions  
Aluminum outer bag 140mm × 200mm  
Pad body 130mm × 180mm
- Weight 23g
- Individual packing One piece / bag

## Target for sale

Target	Purpose of use			
	Disaster storage	Lifesaving tool	Accident correspondence	Treatment assistance
Local government	●	●	●	
Public accommodation	●	●	●	
Public facility, Schools	●	●	●	
Fire department, Police		●		
Armed forces, Garrisons		●	●	●
Sports facility			●	
Construction site, Factory			●	
Hospital, Clinic				●
Nursing home			●	
Animal hospital				●

## Instruction for use

■ Cover a wound with a nonwoven fabric side of the pad and fix it.



## Warnings

- Although this pad is produced for hemostasis, in some cases hemostasis may not be completed according to the injury and the bleeding. You should take the examination of the expert doctor.
- This pad is made to suppress the bleeding temporarily but not to heal the wound. By all means you should take the advice of the expert doctor.
- This pad is disposable. (Reuse is prohibited.)
- Do not use if the sterilized bag is broken or polluted or wet.

## Manufacturing and distribution

Livedo Corporation  
118-1 Kannonbaracho-Otsu, Niihama-shi, Ehime, Japan

## Distributor

MedExport Co.Ltd  
Japan, Kanagawa-Ken, Yokohama-shi  
Nishi-Ku, Chuo 1-36-3 Ark Plaza building  
Tel: +81-70-4495-9875 Email: [info@medexport-japan.com](mailto:info@medexport-japan.com)